


Thermomat Patient warming system

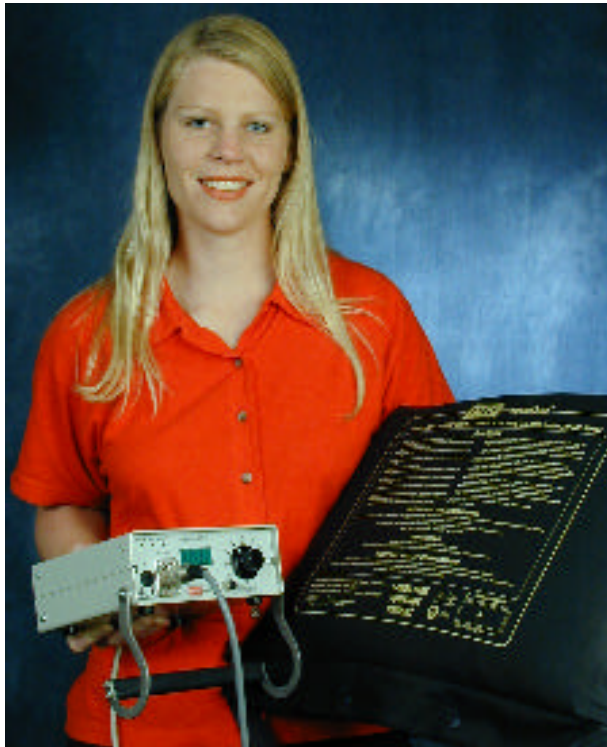

- Versatile and Easy to use
- X-Ray translucent
- Electrically safe
- Padded for extra comfort
- Excellent After Sales Service
- CE marked to comply with EN60601-1 and EN60601-2-35 for medical devices. Council Directive No. 93/42/EEC, Annex III, Section 4

JMW Medical Ltd

Pentland Industrial Estate, Loanhead, Midlothian, EH20 9QH

Tel: 0131 448 0205 Fax: 0131 448 0208

Web: <http://medical.jmw-group.co.uk/> email: medical@jmw-group.co.uk

JMW THERMOMAT patient warming systems are electrically safe, operated on a maximum of 24 volts and isolated from earth, making the JMW system safe even when the mattress is cut or drilled through.

SAFE TEMPERATURE CONTROL, each pad contains a minimum of twelve heat sensors on two circuits. With both visual and audible alarms operating if there is a difference greater than 3 Deg C between each circuit.

COMPATIBLE, The JMW thermomat warming system will not cause any interference with ECG monitors and can be used in conjunction with diathermy.

JMW THERMOMAT PRODUCTS ARE CE APPROVED

The linear carbon heating element avoids leakage and the need for algisides associated with some other pads.

EASY TO USE, a single dial sets the target temperature between 33 Deg. and 40 Deg. C with a push button to confirm the selected temperature. JMW pads are also easy to store, light to carry and clean.

EXCELLENT AFTER SALES SUPPORT

A full service manual backed up by JMW Medical's Technical Department is supplied, or an extended warranty can be arranged.

YOUR JMW CONTROLLER

The T5002 has its temperature set by means of a switch and confirmed by pressing a button. A green LED indicates the unit is on. A flashing amber LED indicates the unit has reached the required temperature. A red LED and an audible alarm indicates a fault or if the pad has been disconnected. Nine LED's are provided on the rear panel for a self test facility and fault diagnosis. The T5002 also has a LED display showing the temperature of the pad.

CHOOSE YOUR JMW PAD

HPA1400 Adult Pad	1400 x 530 mm
HPP900 Pediatric Pad	900 x 530 mm

JMW Medical Ltd

Pentland Industrial Estate, Loanhead, Midlothian, EH20 9QH

Tel: 0131 448 0205 Fax: 0131 448 0208

Web: <http://medical.jmw-group.co.uk/> email: medical@jmw-group.co.uk